eIFU Single Use Instruments

Dokument: GEMMA_eIFU_V04, Stand 10-2022

Manufacturer	GEMMA MEDICAL Service GmbH, Gmuender Strasse 6, 75181 Pforzheim, Germany Tel.: +49 7231 7801215 e-mail: info@gemma-medical.de www.gemma-medical.de	ENGLISH		
Referenced products	REF	Product name / trade name	REF	Product name / trade name
	G10010	Capsulorhexis Forceps	G20051	Phaco Chopper Stop & Chop, angled
	G10012	MICS Capsulorhexis Forceps	G20052	Neuhann Chopper, angled
	G10020	Suturing Forceps, straight	G20070	Push-Pull Hook, angled
	G10030	Tying Forceps straight	G20090	Drysdale Manipulator, angled
	G10031	Tying Forceps, angled	G20110	Lasik – Spatula, double-end
	G10032	Notched Forceps	G20120	Neuhann Chopper – Spatula, double-end
	G10050	IOL Forceps, angled	G20121	Sauder Chopper Push-Pull, double-end
	G20030	Sinsky Hook, angled	G20122	Double-ended Neuhann Chopper – Y-Hook
	G20034	Kuglen Hook	G30010	Barraquer Wire Speculum
	G20040	Spatula 0.50 x 0.30, angled	G30020	Kratz-Barraquer Wire Speculum
	G20041	Y-Hook, angled	G30030	Barraquer Wire Speculum
	G20050	Nagahara Phaco Chopper, angled		
Product description and intended use	<p>The Single Use Ophthalmic Instruments (SUOIs) are surgical instruments intended for ophthalmological-surgical use. The devices are surgically invasive, single use, sterile and are intended for transient use.</p> <p>The SUOIs consists of:</p> <p>Hooks, spatulas: hand-held instruments for the manipulation of ocular tissue or the artificial intraocular lens to be implanted during cataract surgery.</p> <p>Choppers: hand-held instruments for manipulation or sculpting of ocular tissue like the natural lens</p> <p>Forceps: hand-held instruments to grasp or hold ocular tissue or other medical devices such as sutures or IOLs.</p> <p>Speculums: surgical instrument for retraction of the eyelids</p>			
Intended purpose	The Single Use Ophthalmic Instruments (SUOIs) are surgical instruments used in ophthalmological surgeries to hold, grasp, manipulate, sculpting or retract.			
Indications for use	<p>The Single Use Ophthalmic Instruments (SUOIs) are surgical instruments intended for ophthalmological-surgical use. The instruments can be used for any ophthalmic surgery where the anterior chamber will be entered or where the cornea will be treated surgically, like i.e.:</p> <ul style="list-style-type: none"> • Corneal surgeries (i.e. refractive surgeries LASIK, corneal transplantation, DALK, DSAEK, DMEK, etc.) • Lens surgeries (i.e. cataract surgery, RLE, implantable contact lens surgery, etc.) 			
Contraindication	The single use instruments are not intended for use in surgery other than ophthalmology.			
Residual risks/side effects	Risk of cut or puncture injury if handled improperly. No other side effects or residual risks are known.			
Warning	<ul style="list-style-type: none"> • Use before expiration date. • The instruments are delivered sterile and may not be reprocessed or reused. If this product is reprocessed and/or reused, GEMMA MEDICAL cannot guarantee the functionality, material structure, or cleanliness or sterility of the product. Reuse could lead to illness, infection and/or injury to the patient or user. This product is labeled as 'single use' which is defined as a device intended to be used once only for a single patient. • Used surgical instruments might be contaminated. Disease may occur from injury. Dispose in a puncture resistant container. Refer to local regulatory guidelines. 			
Intended user	Only for use by suitably trained and qualified medical professionals			
Intended patient population	No restriction. The suitability of use of the SUOIs with the patient is under the consideration of the ophthalmic surgeon.			
Usage environment	The products may be use in an operation theatre.			
Shelf life	The shelf life of the pre-sterilized instruments is 5 years. See expiration date on the product label.			
Storage	The single use ophthalmic instruments shall be stored a t room temperature in a dry environment.			
Disposal:	Observe country-specific laws and regulations for disposal. To avoid injury, dispose instruments with sharp edges or tips in a protective disposal container designed for sharp instruments.			
Reporting incidents and adverse events	Report all product-related incidents or adverse events immediately to the manufacturer or to the appropriate competent authority.			
Technical Manager	Thomas Diehm			

Symbols used:			
	Expiration Date		Medical Device
	Lot number		Unique Device Identifier
	Reference number		Sterilized using ethylene oxide
	Read instruction for use before use		electronic Instruction For Use, see under www.gemma-medical.de
	Do not reuse		Do not re-sterilize
	Do not use in case of broken package		Handle with care
	Store dry		Keep away from sunlight
	Single sterile barrier system (SBS):		Single sterile barrier system (SBS) with protective packaging outside:
	CE-mark and number of notified body		Quantity

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