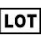
eIFU Single Use Instruments

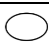
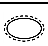
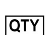
2.2.1_GEMMA_eIFU_V06_DE_EN_ES_IT_FR_NL.docx

 Manufacturer	GEMMA MEDICAL Service GmbH, Gmuender Strasse 6, 75181 Pforzheim, Germany Tel.: +49 7231 7801215 e-mail: info@gemma-medical.de www.gemma-medical.de			ENGLISH																																																				
Referenced products	<table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="width: 15%;">REF</th> <th style="width: 40%;">Product name / trade name</th> <th style="width: 15%;">REF</th> <th style="width: 30%;">Product name / trade name</th> </tr> </thead> <tbody> <tr> <td>G10010</td> <td>Capsulorhexis Forceps</td> <td>G20051</td> <td>Phaco Chopper Stop & Chop, angled</td> </tr> <tr> <td>G10012</td> <td>MICS Capsulorhexis Forceps</td> <td>G20052</td> <td>Neuhann Chopper, angled</td> </tr> <tr> <td>G10020</td> <td>Suturing Forceps, straight</td> <td>G20070</td> <td>Push-Pull Hook, angled</td> </tr> <tr> <td>G10030</td> <td>Tying Forceps straight</td> <td>G20090</td> <td>Drysdale Manipulator, angled</td> </tr> <tr> <td>G10031</td> <td>Tying Forceps, angled</td> <td>G20110</td> <td>Lasik – Spatula, double-end</td> </tr> <tr> <td>G10032</td> <td>Notched Forceps</td> <td>G20120</td> <td>Neuhann Chopper – Spatula, double-end</td> </tr> <tr> <td>G10050</td> <td>IOL Forceps, angled</td> <td>G20121</td> <td>Sauder Chopper – Push-Pull, double-end</td> </tr> <tr> <td>G20030</td> <td>Sinsky Hook, angled</td> <td>G20122</td> <td>Double-ended Neuhann Chopper – Y-Hook</td> </tr> <tr> <td>G20034</td> <td>Kuglen Hook</td> <td>G30010</td> <td>Barraquer Wire Speculum</td> </tr> <tr> <td>G20040</td> <td>Spatula 0.50 x 0.30, angled</td> <td>G30020</td> <td>Kratz-Barraquer Wire Speculum</td> </tr> <tr> <td>G20041</td> <td>Y-Hook, angled</td> <td>G30030</td> <td>Barraquer Wire Speculum</td> </tr> <tr> <td>G20050</td> <td>Nagahara Phaco Chopper, angled</td> <td></td> <td></td> </tr> </tbody> </table>				REF	Product name / trade name	REF	Product name / trade name	G10010	Capsulorhexis Forceps	G20051	Phaco Chopper Stop & Chop, angled	G10012	MICS Capsulorhexis Forceps	G20052	Neuhann Chopper, angled	G10020	Suturing Forceps, straight	G20070	Push-Pull Hook, angled	G10030	Tying Forceps straight	G20090	Drysdale Manipulator, angled	G10031	Tying Forceps, angled	G20110	Lasik – Spatula, double-end	G10032	Notched Forceps	G20120	Neuhann Chopper – Spatula, double-end	G10050	IOL Forceps, angled	G20121	Sauder Chopper – Push-Pull, double-end	G20030	Sinsky Hook, angled	G20122	Double-ended Neuhann Chopper – Y-Hook	G20034	Kuglen Hook	G30010	Barraquer Wire Speculum	G20040	Spatula 0.50 x 0.30, angled	G30020	Kratz-Barraquer Wire Speculum	G20041	Y-Hook, angled	G30030	Barraquer Wire Speculum	G20050	Nagahara Phaco Chopper, angled		
REF	Product name / trade name	REF	Product name / trade name																																																					
G10010	Capsulorhexis Forceps	G20051	Phaco Chopper Stop & Chop, angled																																																					
G10012	MICS Capsulorhexis Forceps	G20052	Neuhann Chopper, angled																																																					
G10020	Suturing Forceps, straight	G20070	Push-Pull Hook, angled																																																					
G10030	Tying Forceps straight	G20090	Drysdale Manipulator, angled																																																					
G10031	Tying Forceps, angled	G20110	Lasik – Spatula, double-end																																																					
G10032	Notched Forceps	G20120	Neuhann Chopper – Spatula, double-end																																																					
G10050	IOL Forceps, angled	G20121	Sauder Chopper – Push-Pull, double-end																																																					
G20030	Sinsky Hook, angled	G20122	Double-ended Neuhann Chopper – Y-Hook																																																					
G20034	Kuglen Hook	G30010	Barraquer Wire Speculum																																																					
G20040	Spatula 0.50 x 0.30, angled	G30020	Kratz-Barraquer Wire Speculum																																																					
G20041	Y-Hook, angled	G30030	Barraquer Wire Speculum																																																					
G20050	Nagahara Phaco Chopper, angled																																																							
Product description and intended use	<p>The Single Use Ophthalmic Instruments (SUOIs) are surgical instruments intended for ophthalmological-surgical use. The devices are surgically invasive, single use, sterile and are intended for transient use. The SUOIs consists of:</p> <p>Hooks, spatulas: hand-held instruments for the manipulation of ocular tissue or the artificial intraocular lens to be implanted during cataract surgery.</p> <p>Choppers: hand-held instruments for manipulation or sculpting of ocular tissue like the natural lens</p> <p>Forceps: hand-held instruments to grasp or hold ocular tissue or other medical devices such as sutures or IOLs.</p> <p>Speculums: surgical instrument for retraction of the eyelids</p>																																																							
Intended purpose	The SUOIs are ophthalmic surgical instruments to hold, grasp, manipulate, sculpting or retract ocular tissues or surgical threads by mechanical effect during cataract, refractive lens exchange and LASIK surgery.																																																							
Indications for use	<ul style="list-style-type: none"> Cataract or refractive lens exchange surgery of adults. The following instruments are indicated for use in the cataract and the similar refractive lens exchange surgery: Forceps: G10010, G10012, G10020, G10030, G10031, G10032, G10050; Manipulators, Hooks, Spatula, Choppers: G20030, G20034, G20040, G20041, G20050, G20051, G20052, G20070, G20090, G20120, G20121, G20122; Speculum: G30010, G30020, G30030 LASIK surgery of adults. The following instruments are indicated for use in Lasik surgery: Spatula double ended: G20110; Speculum: G30010, G30020, G30030 																																																							
Contraindication	The single use instruments are not intended for use in surgery other than ophthalmology.																																																							
Residual risks/side effects	<ul style="list-style-type: none"> Risk of cut or puncture injury if handled improperly. TASS (toxic anterior segment syndrome) 																																																							
Warning and Precautions	<div style="display: flex; align-items: center;">   </div> <ul style="list-style-type: none"> Use before expiration date. Do not use excessive force, because the filigree tips of the instrument can be deformed. Used surgical instruments might be contaminated. Disease may occur from injury. Dispose in a puncture resistant container. Refer to local regulatory guidelines. Scratches on IOL could occur during improper handling of the IOL with forceps G10050. During cataract or refractive lens exchange surgery prevent direct contact for all SUO instruments with other sterilized instruments to potentially avoid TASS. If the instrument comes into contact with potentially contaminated instruments, cross-contamination may occur. 																																																							
Intended user	For use only by ophthalmic surgeons trained and qualified in cataract, refractive lens exchange and LASIK surgery. The use of the SUOIs is restricted to in/outpatient health care facilities.																																																							
Intended patient population	Adult patients requiring <ul style="list-style-type: none"> cataract surgery optional refractive lens exchange or LASIK eye surgery to improve the refractive state of their eye and decrease or eliminate dependency on glasses or contact lenses. Most frequently refractive eye surgeries are used to treat common vision disorders such as myopia, hyperopia, presbyopia and astigmatism. 																																																							
Usage environment	The products may be use in an operation theatre.																																																							
Shelf life	The shelf life of the pre-sterilized instruments is 5 years. See expiration date on the product label.																																																							
Storage	<div style="display: flex; align-items: center;">   </div>																																																							
Disposal:	Observe country-specific laws and regulations for disposal. To avoid injury, dispose instruments with sharp edges or tips in a protective disposal container designed for sharp instruments.																																																							
Reporting incidents and adverse events	Report all product-related incidents or adverse events immediately to the manufacturer or to the appropriate competent authority.																																																							
Technical Manager	Thomas Diehm																																																							
Miscellaneous	A free hard copy version of the user manual can be requested from info@gemma-medical.de.																																																							
Symbols used:																																																								
	Expiration Date		Medical Device																																																					
	Lot number		Unique Device Identifier																																																					

eIFU Single Use Instruments

2.2.1_GEMMA_eIFU_V06_DE_EN_ES_IT_FR_NL.docx

 REF	Reference number	 STERILEEO	Sterilized using ethylene oxide
	Read instruction for use before use	eIFU	electronic Instruction For Use, see under www.gemma-medical.de
	Do not reuse		Do not re-sterilize
	Do not use in case of broken package		Handle with care
	Store dry		Keep away from sunlight
	Single sterile barrier system (SBS):		Single sterile barrier system (SBS) with protective packaging outside:
	CE-mark and number of notified body		Quantity

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